

# GLUTRAL

- **GLUTARALDEHYDE SOLUTION**

**Active Ingredient :**  
**Glutaraldehyde Solution 2.4%**



## HIGH-LEVEL DISINFECTION

GLUTRAL Activated Glutaraldehyde Solution has been used and recognized as a trusted brand for effective high-level disinfection of flexible endoscopes and other medical devices.

**EFFECTIVE** – achieves high-level disinfection in 20 minutes at 20°C.

## MATERIALS COMPATIBILITY

GLUTRAL Activated Glutaraldehyde Solution offers excellent materials compatibility and can therefore be used to disinfect a wide range of medical instruments, made of aluminum, brass, copper, stainless steel, plastics and elastomers.

## MICROBICIDAL ACTIVITY

GLUTRAL Activated Glutaraldehyde Solution provides a wide spectrum efficacy against bacteria, mycobacteria, viruses and fungi. The solution can also achieve some sporicidal activity with longer exposure time.

## TECHNICAL INFORMATION

**SOAK TIME** - High-level Disinfection: 20 minutes at 20°C, Sporicidal activity: 10 hours at 25°C

**USE LIFE**- Up to 14 days

**SHELF LIFE**- 12 months

**OPEN BOTTLE SHELF LIFE**- 14 days (when open and activated)

**STORAGE**- 15-30°C

**DISPOSAL**- Drain or as per hospital policy. Flush thoroughly with water

**SKU's 1ltr, 5ltr**